# NAQI<sup>®</sup> COLD/HOT PACK

## REUSABLE COLD/HOT PACK FOR LONG-LASTING PAIN RELIEF

### INSTRUCTIONS: Cold application & hot application

**Freezer:** Place the NAQI Cold/Hot Pack in the freezer section of refrigerator. Leave it to freeze for at least 2 hours to obtain optimal results. Apply the frozen Cold/Hot Pack to the injured area, which is covered by a dry towel. NAQI Cold/Hot Pack stays flexible at subfreezing temperatures to conform to contours of the treated area. The pack is reusable. Return to the freezing compartment after each use. Use only as directed. The intensity of the cold can be reduced by placing a towel or a non-woven sleeve between the skin and the pack.

**Microwave:** Heat for 3 minutes in a dish filled with water at max. 480 watt and then leave in the microwave to stand for a further 30 seconds. Apply the NAQI Cold/Hot Pack. To re-apply the Cold/Hot Pack immediately, put it back in the microwave for a further 2 minutes. All timings are approximate and depend on microwave output.

**Warm water bath:** Heat the water to 60°C. Leave the NAQI Cold/Hot Pack to soak for 5 min.

**Warning:** Never put the NAQI Cold/Hot Pack directly onto the skin. Avoid contact with open wounds.

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## NAQI<sup>®</sup> COLD PACK

REUSABLE - WIEDERVERWENDBAR - REUTILISABLE - HERBRUIKBAAR

**INSTRUCTIONS:** Place the NAQI Cold Pack in the freezer section of refrigerator. Leave it to freeze for at least 2 hours to obtain optimal results. Apply the frozen Cold Pack to the injured area, which is covered by a dry towel. NAQI Cold Pack stays flexible at subfreezing temperatures to conform to contours of the treated area. The pack is reusable. Return to the freezing compartment after each use.

**Warning:** Use only as directed. Never put the NAQI Cold Pack directly onto the skin. Avoid contact with open wounds.

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NAQI COLD PACK <b>MINI</b>	On face, after eye surgery	7cmx13cm
NAQI COLD PACK <b>SMALL</b>	Gynaecology, Ear, Nose, Throat	8cmx27cm
NAQI COLD PACK <b>STANDARD</b>	Everyday use	13cmx27cm
NAQI COLD PACK <b>SPORT</b>	Remains cold long	13cmx27cm
NAQI COLD PACK <b>MEDIUM</b>	Ankle, Elbow	24cmx27cm
NAQI COLD PACK <b>LARGE</b>	Knee, Shoulder, Back	27cmx37cm